


Instructions for Use

REBOUND[®] PCL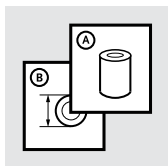

		3
EN	INSTRUCTIONS FOR USE	4
DE	GEBRAUCHSANWEISUNG	7
FR	NOTICE D'UTILISATION	10
ES	INSTRUCCIONES PARA EL USO	13
IT	ISTRUZIONI PER L'USO	16
NO	BRUKSANVISNING	19
DA	BRUGSANVISNING	22
SV	BRUKSANVISNING	25
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	28
FI	KÄYTTÖOHJEET	31
NL	GEBRUIKSAANWIJZING	34
PT	INSTRUÇÕES DE UTILIZAÇÃO	37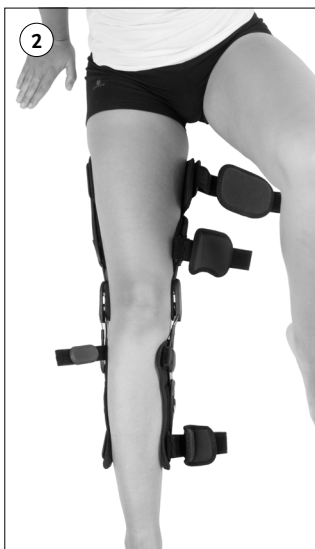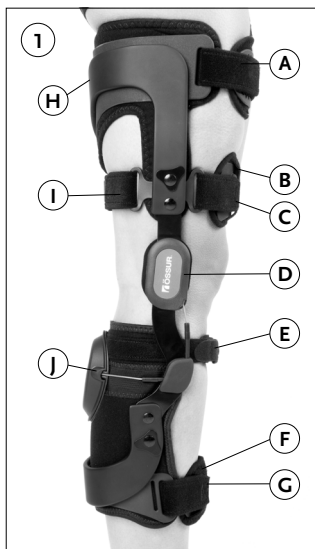

ENGLISH

LIST OF PARTS (Figure 1)

- A. Upper Thigh Strap
- B. Thigh Pad
- C. Lower Thigh Strap
- D. Hinge
- E. AMS Wrap
- F. Shin Pad
- G. Lower Shin Strap
- H. Upper Frame
- I. Rear Thigh Strap
- J. Dynamic Tension System (DTS) with Spool

INDICATION FOR USE

Rehabilitation of posterior cruciate ligament (PCL) injury, whether post-surgical reconstruction or during conservative, non-surgical treatment.

WARNINGS

- The brace must be prescribed and fit by a qualified healthcare provider.
- This brace is intended for post-surgical or post-injury rehabilitation. It is not intended to be used in moderate or high activity and/or impact sports. A qualified healthcare provider must approve the types of activities that the patient may engage in while using the brace.
- This knee brace is a supportive brace only and is not intended or guaranteed to prevent injury; treatment results may vary.
- The patient must not adjust the cable tension. Any adjustments to the cable tension must be made by the healthcare provider. The brace has been designed to be adjusted to the proper tension using the provided shear knobs. Adjustment using any other means can result in excessive tension and injury to the knee.
- In the event of accidental adjustment to the cable tension, the patient should immediately discontinue use of the brace and contact the healthcare provider.
- To avoid discomfort and potential impairment of circulation, the patient should avoid sitting on the frame for an extended period of time.
- If the patient experiences increased pain, swelling, sensation changes, excessive pressure, skin irritation or any adverse reactions while using this brace, immediately discontinue use of the brace and contact the healthcare provider.
- The brace must have an extension stop installed in the hinge during use.
- Keep the leg extended when sitting to increase comfort.

PURCHASER AGREES TO THE FOLLOWING:

1. Össur accepts no responsibility for any injury which might occur while using Össur knee bracing products.
2. With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If the brace causes discomfort or any unusual reactions, discontinue use and contact your health care provider immediately.
3. If you have any questions as to how this knee brace applies to your own specific knee condition, consult your health care provider.
4. This brace is not intended to be used during moderate or high activity

and/or impact sports. If you do not agree to the above conditions, contact your brace provider before using this product.

BRACE APPLICATION

1. Unfasten all straps on the front of the brace (A, C, G) and the AMS wrap (E).
2. The leg must be in full extension when brace is applied. Place the hinge (D) at the center of the knee, as per clinician's instructions. **(Figure 2)**
3. Beginning with the AMS wrap (E) just below the knee, fasten the wrap snugly, as this will aid in brace suspension. **(Figure 3)**
4. To help hold the brace in place, loosely attach the lower thigh strap (C).
5. Next, snugly fasten the bottom strap (G) while centering the shin pad (F) over the bony part of your shin. **(Figure 4)**
6. Fasten the upper thigh strap (A). **(Figure 5)**
7. Fasten strap just above your knee (C) while centering the thigh pad (B). **(Figure 6)**

BRACE MAINTENANCE

The brace is designed for minimum maintenance and care. Follow these easy maintenance steps to keep your brace in good condition.

CLEANING

- Liners, straps and pads may be removed for cleaning.
- Do NOT machine wash
- Do NOT place in dryer
- Hand-wash using mild non-detergent soap
- Rinse thoroughly
- Air-dry

If migration occurs, clean silicone liners with an alcohol wipe or damp cloth. If brace is used in salt water or chlorinated water, rinse well and air dry.

HINGE MAINTENANCE

- Do NOT disassemble the hinge
- Check regularly for foreign materials in hinge (i.e. dirt or grass)
- Carefully clean dirt from hinge using water or compressed air
- Lubricate with lithium grease

ACCESSORIES

The following accessories are available for your PCL brace at an additional charge. To order contact your health care provider.

- PCL Flexion Stop kit
- PCL Extension Stop kit
- Replacement Liners
- Strap kit
- Condyle kit

LIMITED WARRANTY

This warranty covers all defects in materials and workmanship of your new Rebound® PCL Knee Brace. This Warranty covers the frame and hinges for one year from date of sale. The warranty covers all other materials, including straps, sleeves, liners and pads for six months from the date of original purchase.

Össur will repair or replace (at its option) any defective part covered under warranty at no charge. Repairs necessitated by accident, negligence, misuse or abuse by the purchaser are not covered. Also, consequential and incidental damages are not recoverable under this warranty. These limitations may not apply as some states do not allow the exclusion or limitation of incidental or consequential damages. Only the original purchaser is entitled to claim service under this warranty.

To obtain service under this warranty call our Customer Service Department.

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