Bone Growth Stimulators

OL1000 & OL1000SC

Please read before using device

ENGLISH	1
FRANÇAIS	22
DEUTSCH.....	44
ITALIANO.....	66
ESPAÑOL.....	88
NEDERLANDS....	110
ΕΛΛΗΝΙΚΑ.....	132

CAUTION: Federal Law (U.S.A. and Canada) restricts this device to sale, distribution or use by or on the order of a physician.

Table of Contents

Dear Patient,	2
Purpose of the Device	2
Description of the Device.....	3
When the Device Should Not be Used	3
Risks and Benefits	4
General Precautions	5
Directions for Use	5
Starting Each Day's Treatment.....	6
Putting on the OL1000	6
Putting on the OL1000 SC.....	7
Display Screen	8
Checking the Treatment Record	8
Safe Battery Care Information	9
Replacing the Batteries.....	10
Additional Information	11
Device Care and Storage	12
Use While Traveling.....	12
Troubleshooting Information	12
User Assistance Information	15
Customer Care Telephone Numbers.....	15
Appendix A - Shoulder (Clavicle) Use, SC 2/3	16
Appendix B - Upper Leg (Femur) Use, SC 2/3/4	17
Appendix C - Upper Arm (Humerus) Use, SC 2/3	18
Appendix D - Symbols.....	19
Warranty.....	19
Product Specifications.....	20

Dear Patient,

Do not use this product before you read this manual very carefully. If you have questions, please call your doctor or Customer Care. **See page 15 for the Customer Care telephone number in your area.**

Figure 1. OL1000 & OL1000 SC Bone Growth Stimulators

Purpose of the Device

Your doctor has asked you to use the OL1000 or OL1000 SC device (Figure 1, above) for your broken bone that will not heal (called “non-healing” or “nonunion”). When used properly, the device will produce a magnetic field over your broken bone. This field is meant to help the bone heal. To treat yourself, you must wear the device on the outside of your body for 30 minutes per day.

Description of the Device

The OL1000 or OL1000 SC is very easy to use, comfortable to wear, and safe to use. It is designed to create a magnetic field that stimulates your broken bone to heal. You will not feel the magnetic field the device produces when using it. The device has a single “push button” to start your treatment and will automatically shut off after 30 minutes. Each day you will hear one beep when you start treatment and two beeps when your treatment is complete. If you hear beeps before your 30-minute treatment is finished, a picture will be shown on the display screen. Refer to the Troubleshooting Information section on page 13 for help. The device records the number of days that it has been used, which will appear on the display screen. This feature allows you and your doctor to keep track of your care.

When the Device Should Not be Used

- Do not use without the prescription of a doctor.
- Do not use the OL1000 or OL1000 SC if you have a synovial pseudarthrosis (condition where scar tissue forms between the ends of broken bone and prevents healing).
- Do not use the OL1000 or OL1000 SC if you have a heart pacemaker or defibrillator unless your heart doctor has approved it.
- Do not use the OL1000 or OL1000 SC if you have an implant made from magnetic materials at the site of your non-healing broken bone. Almost all implants used today are non-magnetic.

Risks and Benefits

- The safety of the OL1000 or OL1000 SC is not known if you are pregnant or nursing a baby.
- The safety and benefit of the OL1000 or OL1000 SC are not known for people whose bones are still growing (generally 18 years old or less).
- Testing of the OL1000 or OL1000 SC in animals and people has not found any safety problems. However, the chance of safety problems with long time use of this device in people is not known.
- Do not put weight on your broken bone if the bone pieces are able to move.
- If your non-healing broken bone is crooked, note that the OL1000 or OL1000 SC is not intended to help it become less crooked as it heals.
- The safety and benefit of the OL1000 or OL1000 SC are not known for a non-healing broken bone that is caused by a disease rather than an injury.
- Do not use the OL1000 or OL1000 SC if you are unwilling or to follow your doctor's orders or the device instructions.
- If you have bone loss or if your broken bone has been unhealed for a very long time, the OL1000 or OL1000 SC may not work as well.
- The device may not work properly and your treatment may be longer unless you do the following:
 - Always follow your doctor's instructions
 - Always follow your daily treatment schedule
 - Always change the batteries when they need it
 - Always take proper care of the device
- The benefit of the OL1000 or OL1000 SC is that you have a greater chance of healing if you use the device than if you do not use it.

General Precautions

- Do not use the OL1000 or OL1000 SC near products that may have strong magnetic fields, such as audio speakers. The device may not work properly around these products.
- Do not use the OL1000 or OL1000 SC while smoking or near heat, fire or flammable gases because the device may be damaged.
- Do not use the OL1000 or OL1000 SC if there are exposed wires or the device appears damaged.
- Do not modify or repair this device because you may damage it.
- Do not put the device or any of its parts in any liquid.
- Do not drop the device or bend the coils because this may damage it.

Directions for Use

CAUTION: Never use the device if the temperature around you is less than 50°F (10°C) or over 100°F (38°C) or the device may not work properly. The device should be in this temperature range for one hour before treating.

Your doctor or service representative will:

- Show you how to place the device correctly over the spot of your non-healing broken bone;
- Show you how to use the latch or straps to hold the device in place, if needed; and
- Show you how to use the device.

Starting Each Day's Treatment

- Put the device on as you were shown.
- Use the device as you were shown.

OL1000

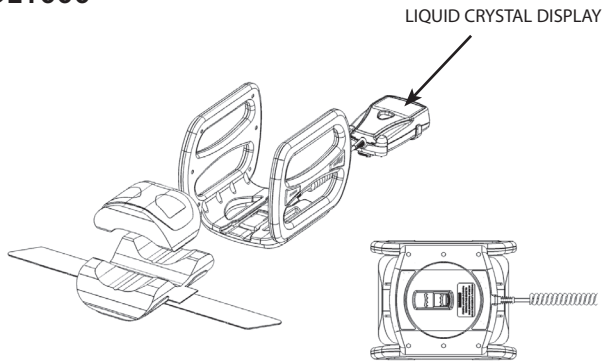
Figure 2.

Putting on the OL1000

1. Place device (Figure 2, above) centered around where your broken bone is located on your body. If you have a cast, brace or external fixator, you may remove the foam pads from the OL1000 for a better fit.
2. Close the strap, if you use one.

Putting on the OL1000 SC

For examples of how to place the OL1000 SC on various areas of the body, please see the Appendices.

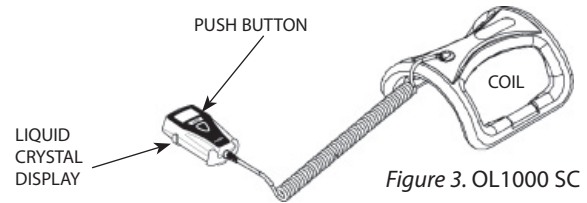
Figure 3. OL1000 SC

1. Center the device's coil around the broken bone. (See illustrations in Appendices.)
2. Close the strap, if you use one.
3. To start a treatment, press the "push button" next to the liquid crystal display (LCD) screen (see Figure 3), **hold it down until it beeps, and then let go.**
 - a. The treatment record will be displayed until it beeps.

- b. The 30-minute treatment countdown will begin.

- c. After 30 minutes, the "treatment complete" icon will appear on the LCD screen, the device will beep twice and it will automatically shut off.

4. Remove the device and store it until the next day. Please see the Device Care and Storage section on page 12 for instructions on the proper storage of the device.

Display Screen

When using the device, the LCD screen will show the time remaining for your daily treatment. The screen may look like this:

In this example, there are 29 minutes and 37 seconds remaining in the 30-minute treatment.

Checking the Treatment Record

You may check your treatment record at any time, except during your daily 30-minute treatment. To view the treatment record, press the “push button” and then let go before it beeps. The LCD screen will look like this:

OL1000 & OL1000SC

The number in the left-hand corner of the treatment record is the number of days you have successfully treated. The number in the upper right-hand corner is the number of days since you first used the device. In this example, it has been 204 days since the device was first used, and 124 treatments have been successfully completed. If the daily treatment has already been completed for that day, and the “push button” is pressed, the treatment record will be shown followed by the “treatment complete” icon. This is a reminder that your daily treatment has already been completed for that day.

Safe Battery Care Information

- Never heat or throw the batteries in a fire.
- Never charge the batteries.
- Never let the battery ends contact each other or other metal. Do not let a piece of metal touch both ends of a battery at the same time.
- Never damage or use damaged batteries.
- Never mix old and new batteries together or use batteries other than in their correct position.
- Always keep batteries at room temperature, about 70°F (20°C) in less than 80% relative humidity and out of direct sunlight.
- Only use the batteries supplied in the carrying case with the OL1000 or OL1000 SC.
- The batteries may be replaced only after the display turns off.

Replacing the Batteries

The LCD screen will show the following picture when it is time for new batteries. This picture will be shown at the start of a treatment or during a treatment. The device will turn off after this picture is displayed.

Never change the batteries when the device is running. Wait until the device stops operating to change the batteries.

Note: Be sure to remove plastic from batteries before replacing them.

Replacing Batteries in the OL1000 & OL1000 SC

The picture in Figure 5, below, shows the correct way to replace the battery. When replacing the battery, you will see the correct way to put them in on the inside of the battery compartment. Use **ONLY** the batteries supplied with the device. They are located in the carrying case.


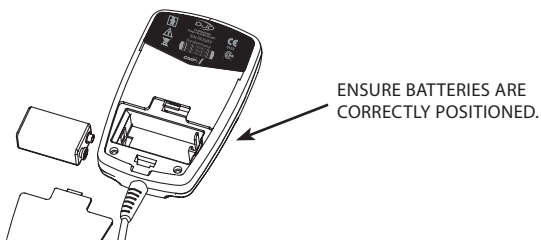
Figure 5. Battery Compartment

Additional Information

- The device can be used **ONLY** 270 days in a row (about nine months). Use the device as long as your doctor tells you to.
- Please take the device with you every time you go to your doctor.
- In clinical studies, the earliest time to heal was three months. The average time to heal was six months. Some non-healing broken bones may take longer to heal.
- There are alternatives to using the OL1000 or OL1000 SC. These include having surgery or using other medical devices. Please ask your doctor if you have any questions about other choices for treating your non-healing broken bone.

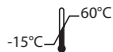Device Care and Storage

- If needed, clean the surface only with a damp cloth.
- Always store the device in its carrying case in a cool, dry place. **Never** keep it where the temperature is less than 5°F (-15°C) or greater than 140°F (60°C). Temperatures below or above could damage the device.
- **Never** store the device or any of its parts in an automobile in cold or hot weather.
- Always store the OL1000 or OL1000 SC in the carrying case provided with the device. The carrying case will protect the device when not in use. It also stores the extra batteries and this manual.
- To avoid the risk of choking or strangulation, always store the OL1000 or OL1000 SC in its carrying case out of reach of small children.

Use While Traveling

You should try to use the OL1000 or OL1000 SC at the same time every day. The clock inside the device counts each day starting at midnight Pacific Standard Time (PST), 8:00 AM Greenwich Mean Time (GMT), or 7:00 AM (GMT) during daylight savings, and will only allow one treatment every 24 hours. If you travel outside of your normal time zone, you should try to treat at the same time you normally would at home. Do not take the OL1000 or OL1000 SC through airport x-ray machines because the x-rays may damage the device.

Troubleshooting Information

If you hear beeps before your 30-minute treatment is complete, there may be a problem with your device. Look at the LCD screen for a picture. Table 1 below shows pictures that may appear on the LCD screen and other common problems (left side of table). The action to take for each picture or problem is on the right side of the table.

Table 1. Display Messages

Display Picture	Action to be Taken
	Magnetic Interference – announced by three beeps. The “X” will flash. 1. If the device looks damaged, call Customer Care. 2. If not, try moving to another location in your house. Make sure you are not near metal objects. 3. If the picture is still displayed, replace the batteries with new ones. 4. If the device still shows the picture, call Customer Care.
	Replace Battery – announced by three beeps. Replace the batteries with new ones.
	Phone For Help – announced by three beeps. Call Customer Care.

(Troubleshooting continues on page 14)

Troubleshooting Information, continued

Problem	Action to be Taken
The device will not turn on, but there is no error message. You may still hear a beep.	<ol style="list-style-type: none"> 1. Try replacing the batteries with new ones. 2. Wait until tomorrow and try treating again. 3. If the device still will not turn on, call Customer Care.
There are lines or bars on the display or the display is too dark to read.	Call Customer Care.
You do not hear any beeps.	<ol style="list-style-type: none"> 1. The beeper may not work if you live where it is very humid. The device will still work. 2. If you do not live where it is humid, listen again tomorrow for beeps. Have somebody else listen too. 3. If the device still does not beep, call Customer Care.
The device appears damaged.	Call Customer Care.
Your broken bone hurts more than it used to.	<ol style="list-style-type: none"> 1. Make sure that you are treating in a comfortable, relaxed position. 2. If you still have more pain than you used to, call your doctor.
You run out of batteries.	Call Customer Care.

A summary is shown below of other pictures that may appear on the LCD screen.

Display Picture	Definition
	Treatment record.
	Time remaining in 30-minute daily treatment.
	Daily treatment successfully completed – announced by two beeps.
	Please wait.

User Assistance Information

For servicing of the OL1000 or OL1000 SC or for further information concerning the use of the device, please contact Customer Care. When your treatment is completed, you should take it to your local recycle center where the unit may be taken in as an electric recycle, like a television or computer. You may also contact Customer Care for assistance with device disposal. The OL1000 and OL1000 SC are not reusable. These devices are intended for single patient use only. They cannot be re-sold or used on multiple patients.

Customer Care Telephone Numbers



United States	800.263.6004
Canada	800.263.6004
Europe	+44 1483.459.659

Clavicle

Appendix A - Shoulder (Clavicle) Use


OL1000 SC Application Instructions. Use for Size 2 & 3

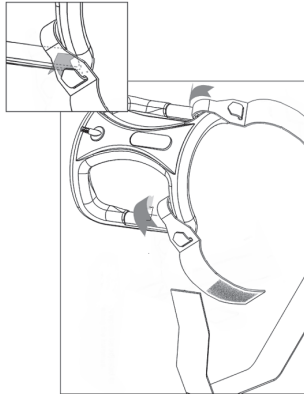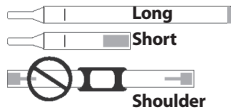Straps

1. Attach the short and long straps to the corners of the device by wrapping the tab end of the strap over the coil, inserting the tab through the open slot, and pulling the tab until tight.

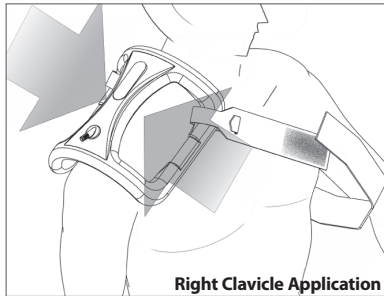2. With the long strap wrapping around the torso, position the coil over the long strap to the short strap.


Right Clavicle Application

3. You may trim the excess strap and re-install the Velcro® hook closer.

Strap Washing Instructions

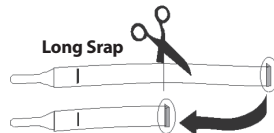
Hand wash with light detergent.
Hang to dry; do not place in dryer


Femur

Appendix B - Upper Leg (Femur) Use

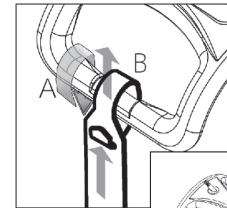
OL1000 SC Application Instructions. Use for Size 2, 3 & 4

1. Attach the short and long straps to the corners of the device by wrapping the tab end of the strap over the coil (A), inserting it through the open slot, and pulling the tab until tight (B).

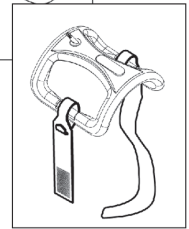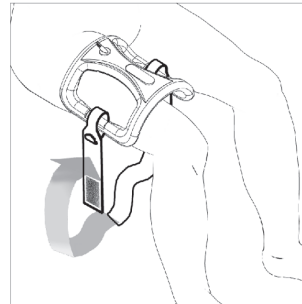
2. With the strap on the inside of the leg, position the center of the coil over the treatment area and secure the long strap to the short strap.

Strap Washing Instructions

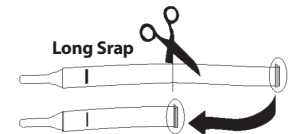
Hand wash with light detergent.
Hang to dry; do not place in dryer


Humerus

Appendix C - Upper Arm (Humerus) Use

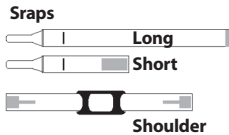OL1000 SC Application Instructions. Use for Size 2 & 3



1. Attach the short and long straps to the corners of the device by wrapping the tab end of the strap over the coil, inserting the tab through the open slot, and pulling the tab until tight.

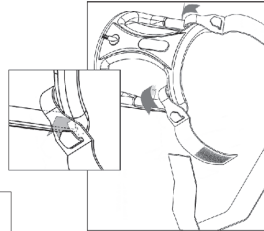2. Attach the shoulder strap to the short and long straps. Adjust the length of the shoulder strap to raise or lower the position of the coil on the arm.

Long strap attaches to the back side.

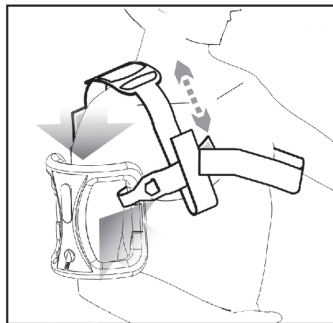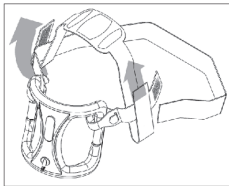
Short strap attaches to the front side.

3. Position the center of the coil over the treatment site

4. Secure the coil in place by wrapping the long strap around the torso and attaching to the shoulder strap.

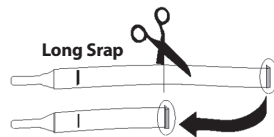5. You may trim the excess strap and re-install the Velcro® hook closer.

Strap Washing Instructions

Hand wash with light detergent. Hang to dry; do not place in dryer

Appendix D - Symbols

EQUIPMENT SAFETY ELECTRIC SHOCK CLASSIFICATION TYPE BF APPLIED PART

CAUTION, CONSULT ACCOMPANYING DOCUMENTS

MANUFACTURER

TEMPERATURE LIMITATIONS

SEPARATE COLLECTION FOR ELECTRICAL AND ELECTRONIC EQUIPMENT - DO NOT DISPOSE OF WITH OTHER HOUSEHOLD WASTE (EUROPE)

Warranty

It is DJO, LLC policy that all complaints about our devices be communicated to Customer Care at 800-263-6004. The Customer Service Representative will provide technical assistance to determine the extent of the problem or educate the patient on the use of the device. If the device cannot be made to function properly, a replacement device is sent to the patient free of charge and the malfunctioning device is brought back for testing and evaluation. This procedure can be done at any time, and as many times as necessary, during the course of treatment. The course of treatment is defined as the nine (9) month period between the time the device is initiated and the Two Hundredth and Seventieth (270th) treatment.

Product Specifications

Manufacturer	DJO, LLC 1430 Decision Street Vista, CA 92081-8553 U.S.A.
Model Types	OL1000, OL1000 SC2, OL1000 SC3, OL1000 SC4
Operating Temperature Range	+50°F (10°C) to +100°F (38°C) Note: CMF by DonJoy Bone Growth Stimulators must remain at Operating Temperature one hour prior to use
Storage and Transportation Temperature Range	+5°F (-15°C) to +140°F (60°C)
Operating Humidity Range	Maximum 85% Relative Humidity
Dimensions	OL1000, 8.4 in x 6.8 in x 7.2 in (21.3 cm x 17.3 cm x 18.2 cm) OL1000 SC4, 13 in x 9.5 in x 13 in (33 cm x 24 cm x 33 cm) OL1000 SC3, 10 in x 7 in x 10 in (25 cm x 18 cm x 25 cm) OL1000 SC2, 8 in x 5 in x 9 in (20 cm x 13 cm x 23 cm)
Weight	OL1000, 2.75 lbs (1.2 kg) OL1000 SC4, 4.9 lbs (2.2 kg) OL1000 SC3, 3.4 lbs (1.5 kg) OL1000 SC2, 2.3 lbs (1 kg)
Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide	
Degree of Protection Against Ingress of Liquids	Ordinary Equipment
Degree of Protection Against Electric Shock	TYPE BF Applied Part
Power Supply	Battery Operated, Replaceable, 9 Volts Direct Current
Mode of Operation	Short time continuous operation, 30 minutes per day
Electromagnetic Compatibility (EMC) Electromagnetic Immunity (EMI)	These products are in conformity with Directive 89/336/EEC

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© 2010 DJO, LLC